

Connect Your E4D Planmeca PlanScan® to BioDent's Workflow



Save time and money when you send scans directly from your E4D Planmeca PlanScan® digital impression scanner. **In addition to all-ceramic restorations, BioDent accepts digital impressions for PFM and full-cast restorations.**

- Go to www.ddxdental.com/login, select "Dental Practice," and enter your login information
- Request that they add BioDent Laboratories to your account.
- Choose "New Case" and use the form to enter the procedure, shade, patient details and any additional notes
- Upload the impression file(s) and submit your case!

If you have any questions, contact BioDent at 800-517-5250 and ask for digital impression technical support.

Looking for more information on our products?

A complete listing of restorative options and material descriptions, preparation and cementation guides and billing codes, visit biodentlaboratory.com/for-dentists

Save money on every digital submission
by choosing model free when possible.

SAVE \$20 On Your First Digital Case

Digital submissions are simple when you choose BioDent. Reference coupon code **GO_DIGITAL** on your Rx when you upload your first case and save \$20 on your first digital submission.

Coupon Code: GO_DIGITAL